

### Technical specifications

- Dimensions: 337 x 271 x 6 mm
- Weight: 560 grams
- Core: lead 2.2 mm
- Cast in: silicone, seamless
- Colour: Red brown RAL 3013
- Disinfection: Up to 136° Celsius



### Delivery/Package:

The package of the Silicone-encased Lead Hand Immobiliser as well as the Hand itself are not disinfected or sterile at the time of delivery.

### First use:

Before first use, the Hand must be disinfected and sterilised.

### Attention:

The user must make sure that the sterility of the Hand up to the opening of the sterile packaging is guaranteed. When in doubt, the Hand is to be submitted for disinfection and sterilisation. Until that time, the Hand may not be used for any medical treatment.

**Optional:** Manual pre-cleaning with cold water, followed by manual pre-cleaning with water of maximum 40°C. Ultrasonic treatment is allowed. Disinfection and sterilisation must follow pre-cleaning.

**Obligated:** The visual control of the Hand on (hair-size) cracks and other damages whereby the lead-core is in sight. As soon as this occurs, the Hand must be rejected and is not suitable for medical treatments anymore.

**Obligated:** A standard wash and disinfection program whereby all sides of the Hand will be reached and a full disinfection of the Hand is guaranteed.

**Obligated:** A standard sterilisation program at a maximum temperature of 136°C. Again, during this program all sides of the Hand must be reached to achieve a complete sterilisation.

**Optional:** Local protocols as long as these do not conflict with the above obligated demands.

**Disclaimer:** The manufacturer and distributor cannot be held responsible for the consequences of not implementing a good cleaning process or the consequences of not following above mentioned measurements.

The Silicone-encased Lead Hand Immobiliser is registered with the Dutch Ministry of health, welfare and support (MVWS) as a medical device class I under number: NL-CA002-2012-24364.

Manufacturer: Dörr

PO Box 8087  
Symonds St  
Auckland 1150  
New Zealand  
[operate@hallmarksurgical.com](mailto:operate@hallmarksurgical.com)

Ph. NZ +64 9 302 0755  
Free Ph NZ 0800 67 37 28  
Ph AUS +61 2 8355 7189  
Free Ph AUS 1800 99 82 13  
[www.hallmarksurgical.com](http://www.hallmarksurgical.com)

**HallmarkSurgical™**  
When you care.